

Version: 02:00

Date of issue: 14/04/2012

Date of revision: 14/08/2017

PALTECH PRECISION 70% IPA Wipes

Description

PALTECH 70% IPA wipes are intended to hygienically clean and disinfect hard surfaces in professional environments.

For professional use only. medical devices within health care environments.

PALTECH 70% IPA wipes are proven to kill most microorganisms such as bacteria, yeasts & viruses. The wipes have been independently tested to a range of EN standards.



Bactericidal		
Organism	*Contact time	Testing standard
Acinetobacter baumannii	30 sec	EN13697
		EN14561
Enterococcus hirae	30 sec	EN1276
		EN13697
		EN13727
		EN14561
		EN16615
Escherichia coli	30 sec	EN1276
		EN13697
MRSA	30 sec	EN13697
		EN13727
		EN14561
Pseudomonas aeruginosa	30 sec	EN1276
		EN13697
		EN13727
		EN14561
		EN16615
Staphylococcus aureus	30 sec	EN1276
		EN13697
		EN13727
		EN14561
		EN16615
VRE	30 sec	EN13697
		EN13727
		EN14561

This data sheet complies with REACH Regulation (EC) N° 1907/2006 - N° 453/2010)
Pal International Limited, Bilton Way, Lutterworth, Leicestershire, LE17 4JA, United Kingdom
Telephone: +44 (0) 1455 555 700 Fax: +44 (0) 1455 555 777 Email: info@palinternational.com

Technical Information

PALTECH

ADVANCED HYGIENE + PROTECTION.
EVERYWHERE.

Version: 02:00

Date of issue: 14/04/2012

Date of revision: 14/08/2017

Tuberculocidal		
Organism	Contact time	Testing standard
Mycobacterium avium	1 min	EN14348
Mycobacterium terrae	1 min	EN14348

Yeasticidal		
Organism	*Contact time	Testing standard
Candida albicans	1 min	EN1650
		EN13624
		EN16615

Virucidal		
Organism	Contact time	Testing standard
Murine Norovirus	1 min	EN 14476
Hepatitis C	5 min	BluTest testing protocol
HIV	5 min	BluTest testing protocol

(*Contact time starting from)